

Contactless Temperature Sensor Instructions for Use



Visit CovidTempSensor.com for more information



1. Hold right wrist approximately 1" away from red dot on the side of the unit.
2. When beeping begins, hold wrist still.
3. Temperature will display in a few seconds. If your temperature is in the **normal range**, a **green indicator light** will appear. If your temperature is **above normal range** a **red indicator light** will appear.
4. If you receive a red indicator light, wait one minute and repeat process.
5. If still red, do not enter the building.
6. If second scan is green, please wait one minute and repeat process. If two out of three scans are red, do not proceed with building entry as noted above.

Since temperature checks are an imperfect measurement to determine COVID-19 or any other disease or ailment, the manufacturer – TTI, Inc. – disclaims any and all responsibility or liability for any harm to anyone resulting from the use of this system.

Technologies Supplied by

Tiny Gecko Evaluation Board– 
PS1240P02CT3 Piezo Buzzer– 

TSD305 IR Sensor– 

Designed and built by the TTI Family of Companies

Designed by 

Components Supplied by   